# pixelbite

Instructions for use

## Scannable VPS bite registration material, fast setting

For powderless 3D data acquisition of the antagonists for the CAD/CAM/CIM technologies (e.g. strip light projection, laser triangulation and CEREC procedures), generates high resolution for artifact-free image reproduction with high brightness and strong contrast. Biting without resistance due to creamy consistency, extremely short time in the mouth (only 45 seconds). Thixotropic, not crumbly, can be cut and milled very well, no breaking out of thin contact point areas.

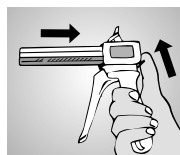
Fig. 1

### 1. Mixing and Dosing

Extrude material from the cartridge with Automix2-system gun (Fig. 1). Place cartridge into mixing gun and remove closure cap by rotation. Eject a small amount of material (bleeding) until equal flow is being extruded from both orifices (Fig. 2). Insert mixing cannula into guide grooves on cartridge and cannula and rotate anti-clockwise to a stop (Fig. 3). Store cartridge with mixing cannula attached for subsequent use.

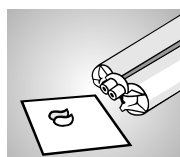
Fig. 2

### 2. Instruction for use

Apply **pixelbite** directly from the mixing cannula, or using a contouring tip, onto the teeth (Fig. 4). Let the registration cure in the required bite position and remove it from the mouth after 45 seconds.

Fig. 3

### 3. Disinfection

After removal from the mouth rinse registrate under lukewarm water. Disinfection for 15 min. in 2 % glutaraldehyde or other usual solutions.


Fig. 4

### Important working hints

- For further work on the registration, observe the instructions of the scan system manufacturer.
- Clean and dry the registration before scanning.
- Remove any retraction astringents or mouth wash solutions prior to the registration, these can influence the setting reaction.
- Do not work with condensation curing silicones.
- Cured silicone materials are chemically stable – avoid staining on clothing.
- Leave no residues of the material in the mouth.
- Only for dental use by qualified personnel.

### Other information:

Silicone impression materials are proven a million times over, unwanted effects cannot be expected in proper use. However, immune reactions, e.g. allergies or irritations cannot be excluded basically. In the case of doubt, we recommend performing an allergy test before application.

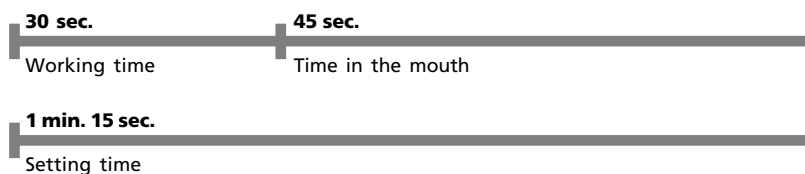### Indications for use:

- Bite registration for powderless 3D data acquisition
- Conventional bite registration
- For coating bite forks and as key material for bearing device registrations
- Fabrication of small model segments

### Technical Data:

- DIN 13903
- **Mixed volume:** 50 ml (cartridge)
- **Mixing ratio:** 1:1
- **Colour code:** base: light grey catalyst: white
- **Mixing time:** omitted (Automix2-system)
- **Working time:** approx. 30 sec.\*
- **Setting time:** approx. 1 min. 15 sec.\*
- **Time in mouth:** approx. 45 sec.
- **Hardness Shore D:** 32
- **Linear dimensional change:** < 0.2 %
- **Application:** At 23 °C ± 2 °C / 73 °F ± 4 °F, 50 ± 5 % rel. humidity, 50 ± 5 % rel. humidity

\* from beginning of mixing at 23 °C ± 2 °C / 73 °F ± 4 °F, 50 ± 5 % rel. humidity. Increased temperatures accelerate, decreased temperatures retard a. m. times.

### Ordering information:

**pixelbite**  
Standard packing **03103**  
2 x 50 ml cartridges  
12 mixing cannulas, pink  
12 contouring tips

Mixing cannulas **02707**  
pink, 48 pcs.

Contouring tips **02648**  
12 pcs.

**greenbite apple**  
Standard packing **02641**  
2 x 50 ml cartridges  
12 mixing cannulas, green  
12 contouring tips

**greenbite colour**  
Standard packing **02207**  
2 x 50 ml cartridges  
12 mixing cannulas, green  
12 contouring tips

**glassbite**  
Standard packing **02571**  
1 x 50 ml cartridge  
4 mixing cannulas, green  
4 contouring tips

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